de

REF Artikelnummer

QTY Menge

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 Herstellungsdatum (JJJJ-MM-TT)

LOT Charge

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FAT Tubing *Liposat*[®] Pro plus (REF. 00003948)

Attention

Do not use the product if the packaging is damaged. Safe use only in combination with the following products:

<i>Liposat</i> [®]		<i>Vibrasat</i> [®]		Accessories	
Product	REF	Product	REF	Product	REF
<i>Liposat</i> [®] Pro plus	00003974	<i>Vibrasat</i> [®] power <i>Vibrasat</i> [®] Pro	00002246 00003920	Cannulas for <i>Vibrasat</i> [®] Manual handle	see brochure 00002254
Table 1		Table 2		Table 3	

Intended use: The FAT Tubing *Liposat*[®] Pro plus tubing set is used both for medical indications, including those accompanied by a change in fatty tissue, and for aesthetic body shaping. The FAT Tubing *Liposat*[®] Pro plus is used to administer tumescent local anaesthesia, other aqueous infusion solutions, as well as endogenous subcutaneous tissue and its components, into the body. The tubing set, sterilised with ethylene oxide, is marketed as a disposable product and may only be used in conjunction with *Liposat*[®] Pro plus.

Single use: The re-use of a single-use product presents a potential risk of infection for the patient or operator. Contaminated products can damage health and cause illness or even death of the patient. Cleaning, disinfection and sterilization can adversely affect essential material properties and product parameters and thus result in product failure.

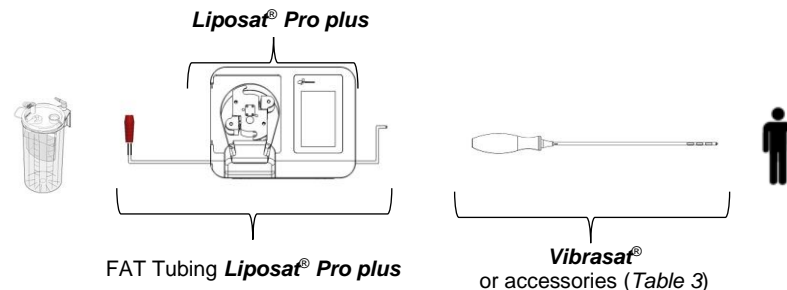
Precautions: Before use, users should be familiar with the *Liposat*[®] pump and *Vibrasat*[®]. Follow the instructions for use of the *Liposat*[®] pumps and *Vibrasat*[®] (see Tables 1 and 2).

Contraindication: See the instructions for use of the *Liposat*[®] pumps (see Table 1).

Quick guide: These instructions are only valid together with the instructions for use for the *Liposat*[®] pumps (see Table 1), *Vibrasat*[®] (see Table 2) and the accessories (see Table 3).

- 1) Open the front cover of the pump.
 - 2) The pump head automatically rotates into a suitable position for inserting the tubing set.
 - 3) Push the tube into the left tube holder until it clicks into place.
Make sure the red funnel of the tube is on the left side. Adjust the tube length so that the connection to the sterile aspiration container is possible.
 - 4) Hold the tubing in position while pressing and holding the button  for inserting the tubing set. You can also turn the rotor by hand in the direction of the arrows.
 - 5) The rotor rotates and threads the tube. Guide the tube by hand in such a way that the rotor can pick it up optimally.
 - 6) Stop the rotor rotation by releasing the button when the tube is in a suitable position.
 - 7) Push the free end of the tube into the right-hand tube holder until it clicks into place.
 - 8) Connect the red funnel of the tubing set to the outlet of the sterile aspiration container.
 - 9) Connect the free end of the tube with one of the infiltration options (Table 2 and 3).
- The tubing set is completely assembled.
- Note that the red funnel is on the left side and the connector for the infiltration options is on the right side.**

Follow the instructions for use of the *Liposat*[®] pumps for handling the infiltration tubing set.

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 Keep away from
sunlight

 Manufacturer

Rx ONLY
Attention:
Under US Federal law
this device may be
sold only to a
physician or ordered
by a physician.

CE 0482

You can find further information on the symbols
used on our homepage:

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Möller Medical GmbH